





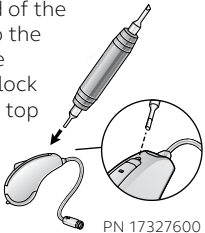
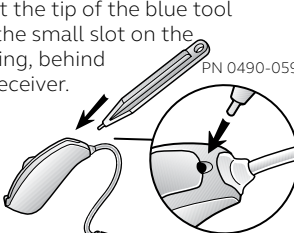

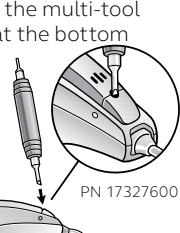
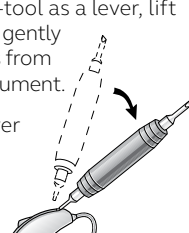
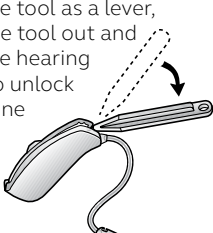
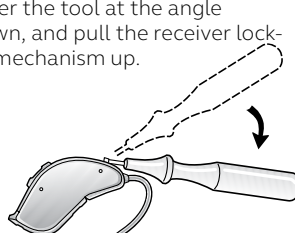
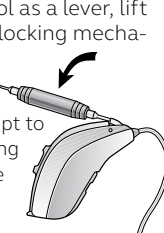
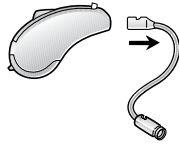
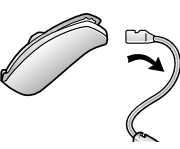
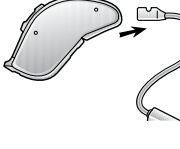
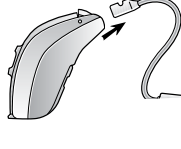
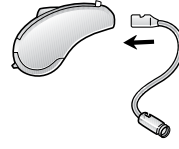
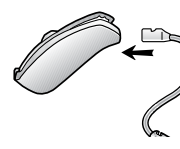
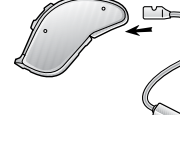
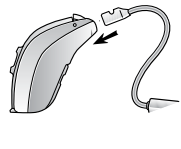
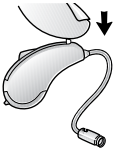
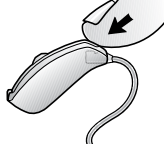
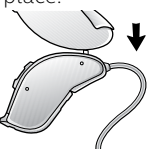
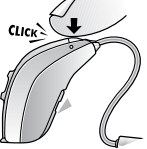
ReSound Sure Fit Receiver Guide

Multiple receiver options enable ReSound Receiver-In-The-Ear (RIE) hearing aids to accommodate a wide range of hearing losses.



Model Battery	61-DRWZ Silver-zinc 312	61 312	62 13	62 312
	ReSound LiNX 3D™ and ReSound LiNX2™			ReSound Enya™
<p>LP Low power receiver</p> 				
<p>MP Medium power receiver</p> 				
<p>HP High power receiver</p> 				
<p>UP Ultra power receiver</p> 				

Changing a Sure Fit receiver

ReSound LiNX 3D™ Rechargeable 61-DRWZ	ReSound LiNX 3D™ and ReSound LiNX2™ 61	ReSound LiNX 3D™ and ReSound LiNX2™ 62	ReSound Enya™ 62
<p>Insert one end of the multi-tool into the opening of the clear receiver lock just above the top microphone.</p>  <p>PN 17327600</p>	<p>Insert the tip of the blue tool into the small slot on the housing, behind the receiver.</p>  <p>PN 0490-059</p>	<p>Insert the tip of the white tool into the small slot in the receiver locking mechanism.</p>  <p>PN 15502500</p>	<p>Insert one end of the multi-tool into the cut-out at the bottom of the locking mechanism.</p>  <p>PN 17327600</p>
<p>Using the multi-tool as a lever, lift the receiver lock gently until it protrudes from the hearing instrument.</p> <p>Note: the receiver lock does not need to be completely removed.</p> 	<p>Using the blue tool as a lever, swing the blue tool out and away from the hearing instrument to unlock the microphone cover.</p> 	<p>Lower the tool at the angle shown, and pull the receiver locking mechanism up.</p> 	<p>Using the multi-tool as a lever, lift the bottom of the locking mechanism to unlock the receiver tube.</p> <p>NOTE: Do not attempt to disconnect the locking mechanism from the hearing instrument.</p> 
<p>Pull the receiver out to remove it from the hearing instrument.</p> 	<p>Pull the receiver down & out to remove it from the hearing instrument.</p> 	<p>Pull the old receiver tube straight out to remove it from the hearing instrument.</p> 	<p>Pull the old receiver tube straight out to remove it from the hearing instrument.</p> 
<p>To insert a new receiver, push the new receiver tube until it is fully seated.</p> 	<p>To insert a new receiver, push the new receiver tube in until it is fully seated.</p> 	<p>To insert a new receiver, push the new receiver tube in until it is fully seated.</p> 	<p>To insert a new receiver, push the new receiver tube in until it is fully seated.</p> 
<p>Lock the receiver tube in place by pushing the clear receiver lock into the hearing instrument until it is flush.</p> 	<p>Lock the receiver tube in place by clicking the locking mechanism closed.</p> 	<p>Lock the receiver tube in place by firmly pushing down the locking mechanism until snugly in place.</p> 	<p>Lock the receiver tube in place by clicking the locking mechanism closed.</p> 
<p>Attach a dome or custom Micromold to the receiver.</p>	<p>Attach a dome or custom Micromold to the receiver.</p>	<p>Attach a dome or custom Micromold to the receiver.</p>	<p>Attach a dome or custom Micromold to the receiver.</p>

Manufacturer according to FDA:

ReSound North America

8001 E. Bloomington Freeway
Bloomington, MN 55420
1-800-248-4327
resoundpro.com

ReSound Government Services

8001 E. Bloomington Freeway
Bloomington, MN 55420
1-800-392-9932
resound.com/governmentservices

Manufacturer according to Health Canada:

ReSound Canada

303 Supertest Road
Toronto, Ontario M3J 2M4
1-888-737-6863
resoundpro.com

